

<b>Composition</b>	A sterile solution containing ethanol 70% v/v in purified water.
<b>Actions</b>	Alcohol 70% is bactericidal to most vegetative organisms. It is not effective against viruses and spores. It acts relatively quickly to denature cellular proteins of microorganisms and lower their surface tension thus facilitating their removal.
<b>Use</b>	<p>Alcohol 70% is a bactericidal surface spray disinfectant - hospital grade. It is suitable for surface disinfection of critical areas such as laminar flow cabinets and of materials to be used in critical areas such as aseptic dispensing of pharmaceuticals.</p> <p>It is also a general purpose disinfectant for hospital equipment or surfaces not likely to come into contact with broken skin. Do not use on Therapeutic Devices.</p>
<b>Directions for use</b>	<p>Use undiluted. Clean surfaces first, the disinfectant may not be effective if the surface is not pre-cleaned. The spray trigger nozzle must be set to produce a spray zone (circle of 20cm diameter). Spray two times on to the surface to be disinfected ensuring the nozzle is 30cm from the surface. Leave 1 minute and allow to dry.</p> <p>The enclosed bottle and its contents are sterile if the outer bag is intact. Sterility cannot be guaranteed if the outer bag is damaged.</p> <p>Do not mix with detergents or other chemicals.</p>
<b>Precautions</b>	Not to be swallowed. Flammable do not use near a naked flame. Keep out of reach of children.
<b>Safety Directions</b>	Avoid contact with eyes and mouth.
<b>First Aid</b>	<p>For advice, contact a Poisons Information Centre (phone Australia 131126) or a doctor.</p> <p>Give plenty of water to drink.</p> <p>If in eyes hold eyes open and flood with water for at least 15 minutes and see a doctor.</p>
<b>Storage</b>	Store below 25°C.
<b>Poison Schedule</b>	Not scheduled.

Code	Product	Size	Carton
ALC 01602 F	ALCOHOL 70% SURFACE SPRAY (Plastic Spray Bottle)	500mL	20

**Contact Point**

Drug Information Pharmacist  
 ORION Laboratories Pty Ltd ABN 56 009 293 136  
 25-29 Delawney Street, Balcatta, Western Australia 6021 AUSTRALIA  
**Telephone (all hours): +618 9441 7800**  
 FREE PHONE: 1800 805 546 FREE FAX: 1800 004 110  
 EMAIL: customerservice@orion.net.au; WEBSITE: www.orion.net.au  
 ORION® is a registered trademark of Orion Laboratories Pty Ltd

**ALCOHOL 70% SURFACE SPRAY**

---

Prepared by: Anna McLean	Date: November 2009
Checked by: Robert Kimpton	Date: November 2009

## Revision Information:

Date	Document	Superseded Document	Revision Information
April 2006	ALC01602_1 April 2006	PROPOSED USER GUIDE alcohol surface spray_1_UG January 2006	Allocation of product codes.
May 2009	ALC01602_02_ May 2009	ALC01602_1 April 2006	General review
November 2009	ALC01602_03 November 2009	ALC01602_02 May 2009	<i>First Aid</i> : Inclusion of phone number