

<b>Composition</b>	A safe and colourless, water soluble lubricating gel for personal and surgical use. Clinigel contains propylene glycol BP and hydroxyethylcellulose BP. It is preserved with chlorocresol BP 0.1%
<b>Use</b>	Clinigel has lubricating properties and is intended to be used for personal lubrication and surgical lubrication of non-sterile mucosal surfaces and orifices.
<b>Contraindications</b>	The main contraindication is to known hypersensitivity to propylene glycol and chlorocresol.
<b>Directions for Use</b>	Apply to instruments or directly to mucosal surfaces and orifices as required.
<b>Precautions</b>	Propylene glycol was neither a reproductive nor a developmental toxicant in animal ingestion studies.
<b>Adverse Reactions</b>	Clinigel is normally well tolerated, but should skin irritation occur discontinue use.  After topical use, propylene glycol may produce some local irritation, particularly if applied under occlusive dressings or to mucous membranes; toxicity may occur following application to burns. Hypersensitivity reactions have also been reported.
<b>Storage</b>	Store below 30°C.
<b>Presentation</b>	ARTG 145353  Clinigel is supplied in a 50g tube.

Code	Product	Size	Carton
CLI 01801 F	Clinigel	50g	10

**Contact Point**  
Drug Information Pharmacist  
ORION Laboratories Pty Ltd ABN 56 009 293 136  
25-29 Delawney Street, Balcatta, Western Australia 6021 AUSTRALIA  
**Telephone (all hours): +618 9441 7800**  
FREE PHONE: 1800 805 546 FREE FAX: 1800 004 110  
EMAIL: customerservice@orion.net.au; WEBSITE: www.orion.net.au  
ORION® is a registered trademark of Orion Laboratories Pty Ltd

Prepared by: Anna McLean	Date: January 2011
Checked by: Robert Kimpton	Date: January 2011

Date	Document	Superseded Document	Revision Information
June 2006	CLI01801_02 June 2006	CLI01801_01 July 2003	General Review; change of address
November 2007	CLI01801_03 November 2007	CLI01801_02 June 2006	Changes in all Sections
January 2011	CLI01801_04 January 2011	CLI01801_03 November 2007	General review

END OF USER GUIDE